




## TAKING A MEASUREMENT

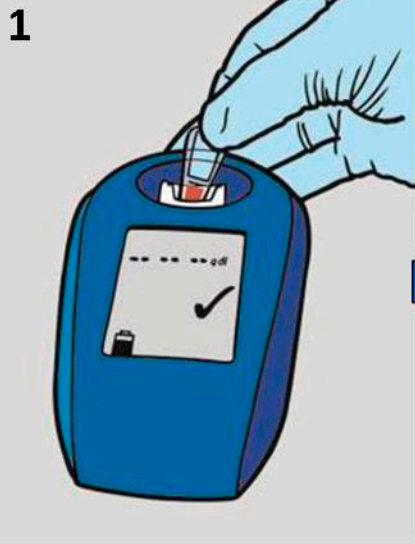


Place the filled cuvette into the cuvette holder.

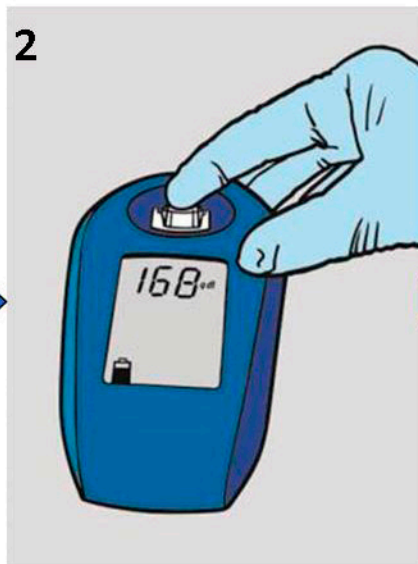

Press down firmly until you hear a click and the light flashes, then release pressure and the haemoglobin value will be displayed instantly.

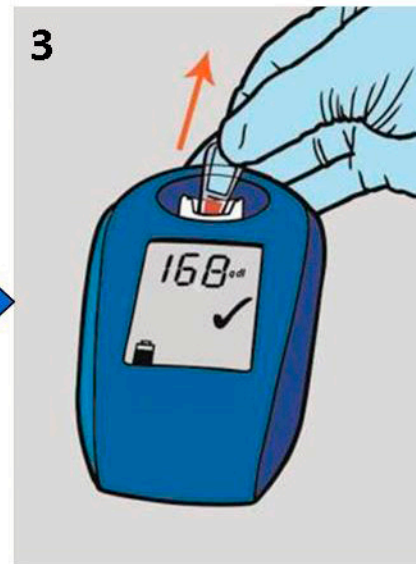

Withdraw the cuvette immediately.  
Record the reading as soon as the ✓ appears.

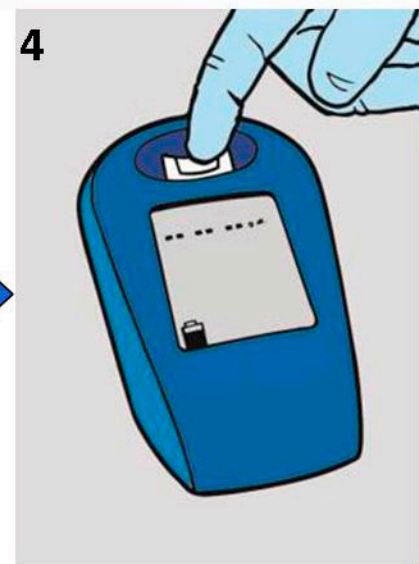

To reset the analyser or to erase a previous reading, simply make a 'blank' measurement by pressing down the empty cuvette holder.

If the cuvette has been left in the holder for too long, an E03 error code will appear. Simply reset the analyser by performing a 'blank' measurement as detailed in 4.